GB ENZYMAX EARTH POWDER PAX (IMS-1332, IMS-1333, IMS-1332SP)

Application: Enzymatic Cleaner on Powder base

Net Contents: IMS-1332: 32 Powder packets 14 g (water-soluble) - IMS-1333: 96 Powder packets 14 g (water-soluble) - IMS-1332SP: 3 Powder packets 14 g (water-soluble)

Instructions for use: Wear gloves and eye protection. Use with water 32°– 60°C, demineralized water is ideal. **Presoak:** Add 1 packet per 3.8L of water. Soak for 3-5 minutes. **Linen Presoak:** Add 1 to 2 packets per 3.8L. **Ultrasonic Cleaning:** Add 1 packet per 3.8L of water. Rinse thoroughly, dry and proceed with sterilization. **Evacuators:** Add 2 packets per 3.8L of water. Insert evacuator hose, turn system on full open for one minute. Suggested maintenance twice weekly. Do not use disinfectants together with Enzymax. - Do not use Enzymax in the Thermodesinfector unit. Change solution at least once a day.

UFI: 96EK-8WYE-TAOH-AXRM

DANGER

H315 - Causes skin irritation. **H318** - Causes serious eye damage. **H334** - May cause allergy or asthma symptoms or breathing difficulties if inhaled. **P261** - Avoid breathing dust/fume/gas/mist/vapours/spray. **P280** - Wear protective gloves, protective clothing, eye protection, face protection. **P302+P352** - IF ON SKIN: Wash with plenty of water. **P304+P340** - IF INHALED: Remove person to fresh air and keep comfortable for breathing. **P305+P351+P338** - IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **P310** - Immediately call a POISON CENTER or doctor. **Safety Information:** EUH208 Contains: Subtilisin; Sodium dodecylbenzenesulfonate; Sodium metasilicate. May produce an allergic reaction. Dispose of contents/container in accordance with local/regional/national/international regulations.

SDS available on request or at: www.hu-friedy.eu

24H Emergency number: INTERNATIONAL +49 - (0) 6132 - 84463, GBK GmbH (24h - 7d/w - 365d/a) / In England and Wales: NHS Direct - 0845 4647, In Scotland: NHS 24 - 08454 24 24 24

IT ENZYMAX EARTH POWDER PAX (IMS-1332, IMS-1333, IMS-1332SP)

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