GB ENZYMAX LIQUID CONCENTRATE (IMS-1222; IMS-1222C)

Application: Enzymatic Cleaner liquid

Net Contents: IMS-1222: Box of 40 Liquid Packets of 10 ml (concentrate)

Instructions for use: Shake well before using. Wear gloves and eye protection. Use with water 32°–60°C, demineralized water is ideal. **Presoak:** Add 30ml of concentrate per 3.8 L of water. Bottles with pump insert: pump one time per liter of water. Single packet: 1pck (10 ml) per 3.5 liter of water. Soak for 3–5 minutes. Rinse, dry thoroughly and proceed with sterilization. **Ultrasonic Cleaning:** Add 30 ml of concentrate per 3.8 L of water. Bottles with pump insert: pump one time per liter of water. Single packet: 1pck (10ml) per 3.5 liter of water. After cleaning, rinse, dry thoroughly and proceed with sterilization. **Evacuators:** Add 60 ml per 3.8 L of water. Insert evacuator hose, turn system on full open for one minute. Suggested maintenance twice weekly.

- Do not use disinfectants together with Enzymax. - Do not use Enzymax in the Thermodesinfector unit. Change solution at least once a day

UFI: 5V1X-9665-UH3D-2SRJ

! DANGER

H318 - Causes serious eye damage. **H334** - May cause allergy or asthma symptoms or breathing difficulties if inhaled. **H412** - Harmful to aquatic life with long lasting effects. **P261** - Avoid breathing dust/fume/gas/mist/vapours/spray. **P273** - Avoid release to the environment. **P280** - Wear protective gloves, protective clothing, eye protection, face protection. **P304+P340**

- IF INHALED: Remove person to fresh air and keep comfortable for breathing. **P305+P351+P338**
- IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **P310** - Immediately call a POISON CENTER or doctor. **Safety Information:** Contains Subtilisin, Amylase, alpha- May produce allergic reaction.

SDS available on request or at: www.hu-friedy.eu

24H Emergency number: INTERNATIONAL +49 - (0) 6132 - 84463, GBK GmbH (24h - 7d/w - 365d/a) / In England and Wales: NHS Direct - 0845 4647, In Scotland: NHS 24 - 08454 24 24 24

FR ENZYMAX LIQUID CONCENTRATE (IMS-1222; IMS-1222C)

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IT ENZYMAX LIQUID CONCENTRATE (IMS-1222; IMS-1222C)

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NL ENZYMAX LIQUID CONCENTRATE (IMS-1222; IMS-1222C)

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