GB ENZYMAX LIQUID (IMS-1226)

Application: Enzymatic Cleaner liquid

Net Contents: IMS-1226: 3,8 liters bottle

Instructions for use: Shake well before using. Wear gloves and eye protection. Use with water 32°–60°C, demineralized water is ideal. **Presoak:** Add 30ml of concentrate per 3,8L of water. Bottles with pump insert: pump one time per liter of water. Single packet: 1pck (10ml) per 3,5 liter of water. Soak for 3–5 minutes. Rinse, dry thoroughly and proceed with sterilization. **Ultrasonic Cleaning:** Add 30ml of concentrate per 3,8L of water. Bottles with pump insert: pump one time per liter of water. Single packet: 1pck (10ml) per 3,5 liter of water. After cleaning, rinse, dry thoroughly and proceed with sterilization. **Evacuators:** Add 60ml per 3,8L of water. Insert evacuator hose, turn system on full open for one minute. Suggested maintenance twice weekly.

- Do not use disinfectants together with Enzymax. - Do not use Enzymax in the Thermodesinfector unit. Change solution at least once a day

UFI: UKDK-RW4N-PA02-PJP5

DANGER

H319 - Causes serious eye irritation. **H334** - May cause allergy or asthma symptoms or breathing difficulties if inhaled. **P261** - Avoid breathing dust/fume/gas/mist/vapours/spray. **P280** - Wear protective gloves, protective clothing, eye protection, face protection. **P304+P340** - IF INHALED: Remove person to fresh air and keep comfortable for breathing. **P305+P351+P338** - IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **P314** - Get medical advice/attention if you feel unwell. **Safety Information:** Contains Subtilisin - May produce allergic reaction.

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