GB IMS-1455 Shine reNEW Instrument Wipes

20 wipes: clean, polish and protect instruments. Removes stains, rust, water deposits and tarnish. Safe for use on: stainless steel, brass, copper and anodized aluminum.

Instructions for use: Open lid, squeeze the canister and pull lid directly up to remove entire lid from canister. Peel off seal and discard. Locate wipe at the center of the roll, twist to a point and thread through small opening in lid. Replace lid and pull wipe through. Always snap lid securely shut between uses to prevent moisture loss. To remove stains, wearing gloves rub affected area with cloth until stain is no longer visible. For heavily stained spots, rub area with greater pressure. After use, wash and sterilize instrument as per normal instrument processing. Do not use on the rings of gold or black-handled instrument (tungsten carbide or SuperCut).

UFI: JFEK-TWIM-RA00-9YGT

DANGER

H318 Causes serious eye damage. **P101** If medical advice is needed, have product container or label at hand. **P102** Keep out of reach of children. **P280** Wear protective gloves/protective clothing/eye protection/face protection. **P305+P351+P338** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **P310** Immediately call a POISON CENTER/doctor. Dispose of contents/container in accordance with local/regional/national/international regulations. **Safety Information:** EUH208 Contains: Alcohols,

C12-14-secondary, ethoxylated **SDS** available on request or at: www.hu-friedy.eu
24H Emergency number: INTERNATIONAL +49 - (0) 6132 - 84463, GBK GmbH (24h - 7d/w - 365d/a) / In England and Wales: NHS Direct - 0845 4647, In Scotland: NHS 24 - 08454 24 24 24

FR IMS-1455 Shine reNEW Instrument Wipes

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IT IMS-1455 Shine reNEW Instrument Wipes

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