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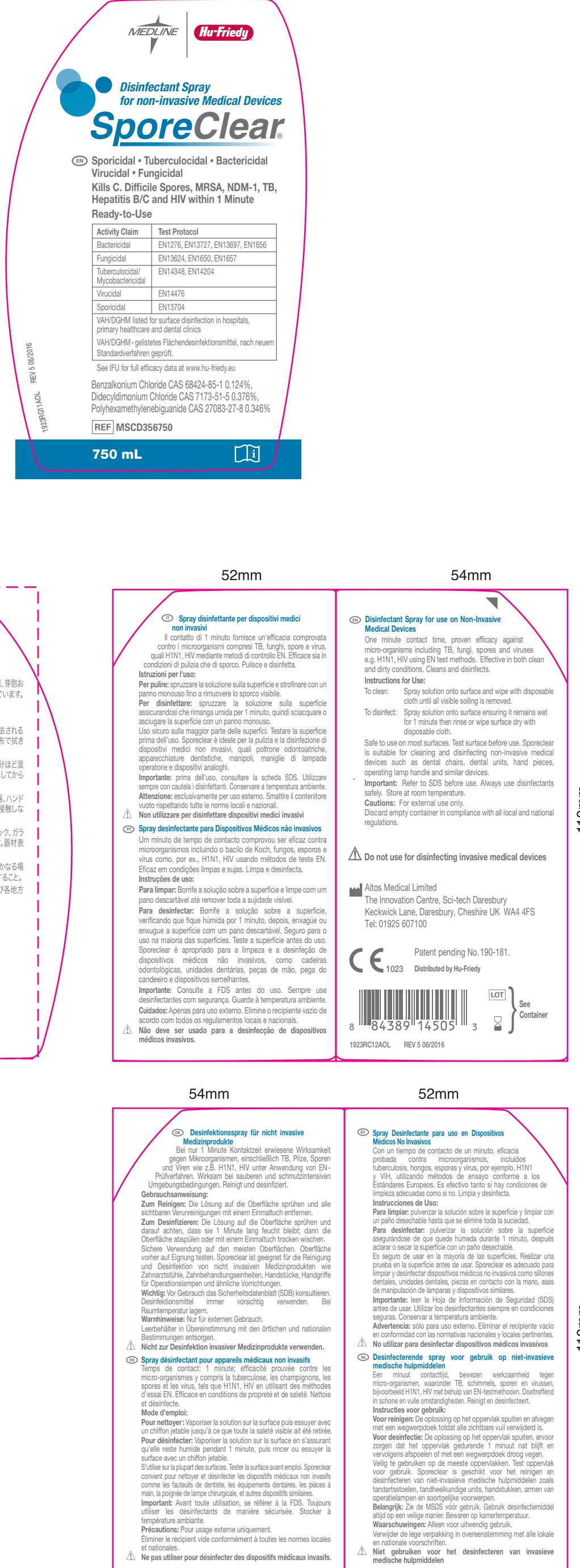
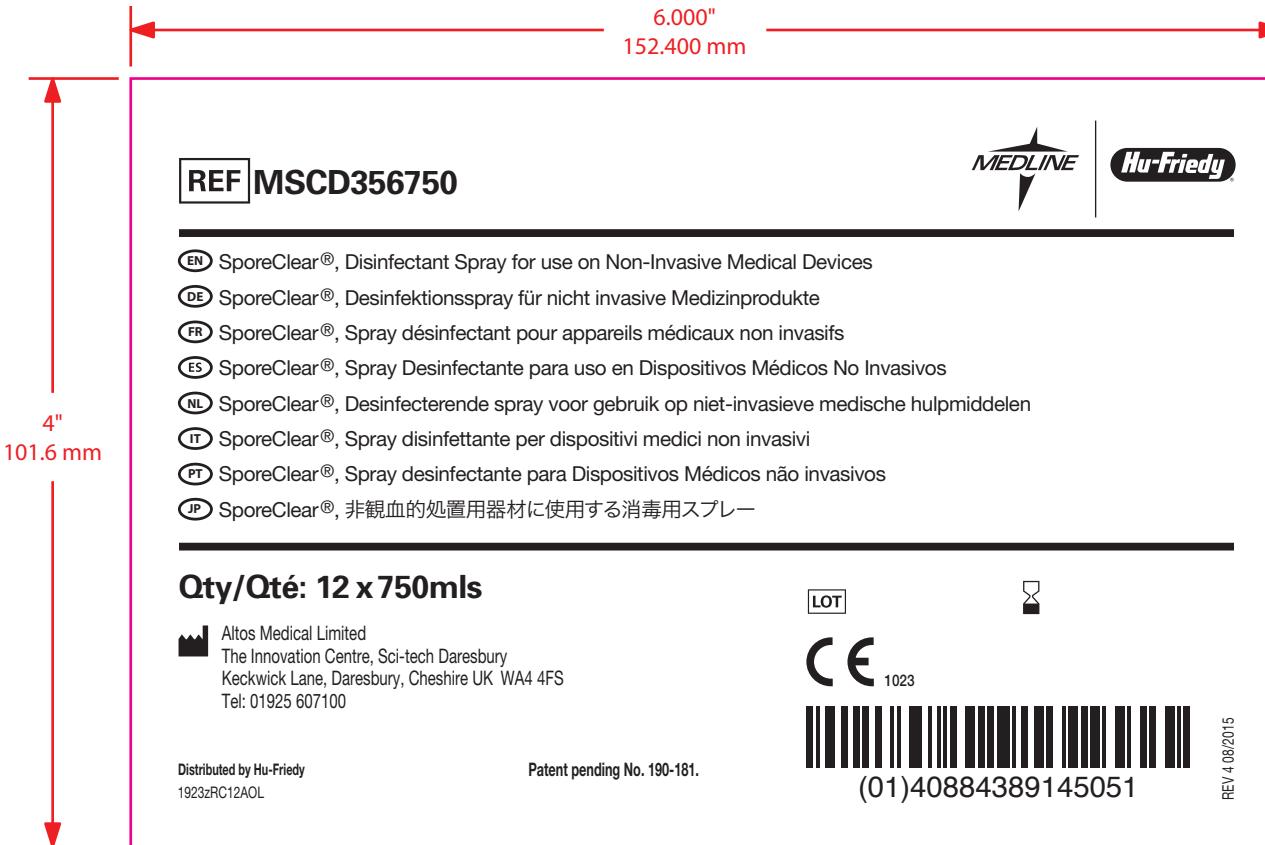
File Name: MSCD356750_1_A0L.ai
Designer: Lolita
Date Started: 3/26/12
Date Finished:

PMS COLORS USED CMYK

	307	Cool Gray 11	1945

Revision **F**

DE Disinfectant Spray for use on Non-Invasive Medical Devices

One minute contact time, proven efficacy against micro-organisms including TB, fungi, spores and viruses e.g. H1N1, HIV using EN test methods. Effective in both clean and dirty conditions. Cleans and disinfects.

Instructions for Use:

To clean: Spray solution onto surface and wipe with disposable cloth until all visible soiling is removed.

To disinfect: Spray solution onto surface ensuring it remains wet for 1 minute then rinse or wipe surface dry with disposable cloth. Sporeclear is suitable for cleaning and disinfecting non-invasive medical devices such as dental chairs, dental units, hand pieces, operating lamp handle and similar devices. Safe to use on all non-porous materials including stainless steel, vinyl, plastics, glass, laminate etc. Test surface before use.

Important: Refer to SDS before use. Always use disinfectants safely. Store at room temperature.

Cautions: For external use only. Discard empty container in compliance with all local and national regulations.

DE Do not use for disinfecting invasive medical devices

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Altos Medical Limited