165 x 285mm

(EN) Disinfectant Wipes for use on Non-Invasive Medical Devices
One minute contact time, proven efficacy against micro-organisms including TB, fungi, spores and viruses e.g. H1N1, HIV using EN test methods. Effective in both clean and dirty conditions. Cleans and disinfects.
Instructions for Use. To clean: Wipe surface until all visible soiling is removed. To disinfect: Wipe surface ensuring it remains wet for 1 minute. Safe to use on most surfaces. Sporeclear is suitable for cleaning and disinfecting non-invasive medical devices such as dental chairs, dental units, hand pieces, operating lamp handle and similar devices. Safe to use on all non-porous materials including stainless steel, vinyl, plastics, glass, laminate etc. Test surface before use. **Important:** Refer to SDS before use. Always use disinfectants safely. Dispose of used wipe immediately after use. Do not dispose of wipes in macerators or down the toilet. Store at room temperature. **Cautions:** For external use only. Discard empty container in compliance with all local and national regulations.
⚠️ Do not use for disinfecting invasive medical devices.

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Disinfectant Wipes for non-invasive Medical Devices

SporeClear®

EN Sporicidal • Tuberculocidal • Bactericidal

Virucidal • Fungicidal

Kills C. Difficile Spores, MRSA, NDM-1, TB and Hepatitis B/C and HIV within 1 Minute
Ready-to-Use

Activity Claim	Test Protocol	Contact Time
Bactericidal	EN1276, EN13727, EN13697, EN1656	1 minute
Fungicidal	EN13624, EN1650, EN1657	1 minute
Tuberculocidal/ Mycobactericidal	EN14348, EN14204	1 minute
Virucidal	EN14476	1 minute
Sporicidal	EN13704	1 minute
VAH/DGHM listed for surface disinfection in hospitals and primary healthcare		

See IFU for full efficacy data at www.hu-friedy.eu

Benzalkonium Chloride CAS 68424-85-1 0.310%,
Didecyldimonium Chloride CAS 7173-51-5 0.940%,
Polyhexamethylenebiguanide CAS 27083-27-8 0.866%

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Altos Medical Limited