EN Disinfectant Concentrate for use on Non-Invasive Medical Devices

One minute contact time, proven efficacy against micro-organisms including TB, fungi, spores and viruses e.g. H1N1, HIV using EN test methods. Effective in both clean and dirty conditions. Cleans and disinfects.

Instructions for Use: Dilute 40mL of Sporeclear Concentrate in 460mL of water.

To clean: Apply to surface then remove all visible soiling with disposable cloth.

To disinfect: Apply to surface ensuring it remains wet for 1 minute, then rinse or wipe surface dry with a disposable cloth. Safe to use on most surfaces. Test surface before use. Sporeclear is suitable for cleaning and disinfecting non-invasive medical devices such as dental chairs, dental units, hand pieces, operating lamp handle and similar devices. **Important:** Refer to SDS before use. Always use disinfectants safely. Store at room temperature.

DA Do not use for disinfecting invasive medical devices

Danger: Causes skin irritation. Causes serious eye damage. May cause an allergic skin reaction. Suspected of causing cancer. May cause damage to organs through prolonged or repeated exposure. Very toxic to aquatic life with long lasting effects. **Avoid release to the environment.** Wear protective gloves/protective clothing/eye protection/face protection. If on skin, wash with plenty of water/soap. If in eyes, rinse cautiously with water for several minutes, remove contact lenses if present and easy to do, continue rinsing. If exposed or concerned, get medical advice/attention. Dispose of contents in accordance with local, state or national legislation.

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LOT } See Containar

Patent pending No. 190-181.
Distributed by Hu-Friedy

RG11AOL REV 5 06/2016

File Name: MSCW35C01_1_AOL.ai PROD MGR APPROVED APPD W/CHANGES REPROOF
Designer: Danielle Carlin Prod MGR signature verifies artwork AND dielines are correct
Date Started: 5/16/11
Date Finished: 6/30/11

PMS COLORS USED CMYK Revision: FINAL
1945 Cool Gray 11 307 151

Disinfectant Concentrate for use on Non-Invasive Medical Devices

• Sporidical • Tuberculocidal • Bactericidal • Virucidal • Fungicidal
Kills C. difficile Spores, MRSA, NDM-1 TB, Hepatitis B/C and HIV within 1 minute.
1 minute contact time proven efficacy against microorganisms including TB, fungi, spores and viruses e.g. H1N1, HIV using EN test methods. Effective in both clean and dirty conditions. Cleans and disinfects.

Instructions for Use: Dilute 40mL of Sporeclear Concentrate in 460mL of water. **To clean:** Apply to surface then remove all visible soiling with disposable cloth.

To disinfect: Apply to surface ensuring it remains wet for 1 minute, then rinse or wipe surface dry with a disposable cloth. Safe to use on most surfaces. Sporeclear is suitable for cleaning and disinfecting non-invasive medical devices such as dental chairs, dental units, hand pieces, operating lamp handle and similar devices. Safe to use on all non-porous materials including stainless steel, vinyl, plastics, glass, laminate etc. Test surface before use.

Important: Refer to SDS before use. Always use disinfectants safely. Store at room temperature.
Danger: Causes skin irritation. Causes serious eye damage. May cause an allergic skin reaction. Suspected of causing cancer. May cause damage to organs through prolonged or repeated exposure. Very toxic to aquatic life with long lasting effects.

Avoid release to the environment. Wear protective gloves/protective clothing/eye protection/face protection. If on skin, wash with plenty of water/soap. If in eyes, rinse cautiously with water for several minutes, remove contact lenses if present and easy to do, continue rinsing. If exposed or concerned, get medical advice/attention. Dispose of contents in accordance with local, state or national legislation.

Do not use for disinfecting invasive medical devices

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Altos Medical Limited