WASHER-DISINFECTOR CLEANING MONITOR

For Use in Routine Monitoring of the Wash Phase
in Washer/Disinfectors

②

INDICATIONS FOR USE:

The single use Hu-Friedy Washer-Disinfectant Cleaning Monitor is designed for monitoring cleaning efficiency during the wash process.

INSTRUCTIONS FOR USE:

1. Insert a Washer-Disinfectant Cleaning Monitor strip (IMS-1200W) into Cleaning Monitor Holder (IMS-1200H) completely. The front side of the monitor with red test soil will be visible in the holder window, as shown below.

2. Position the Cleaning Monitor Holder face up inside a basket. Place basket on a shelf in the washer. Vary positions and shelves of Washer-Disinfectant Cleaning Monitor placement cycle to cycle.
3. Run a complete washer cycle. We recommend monitoring every shelf on every washer each day for a machine release and using a minimum of one Washer-Disinfectant Cleaning Monitor strip per load for routine monitoring throughout the day.
4. At the end of the washing cycle, remove the Washer-Disinfectant Cleaning Monitor from the holder and read the result. Please see visual interpretation of results below. Caution: Holder and monitor strip may still be hot! Remove with care.

INTERPRETATION OF MONITOR RESULTS:

Untreated Monitor, or critical/poor result

Insufficient result

Optimum result

5. If desired, you may also record the location of monitor placement in the washer-disinfectant on the strip itself with a fine tip waterproof marker. Place a hash mark or circle representing the position of the Wash-Checks on the location rectangle that simulates an imaginary washer basket. This allows for placement tracking.
6. If desired, staple the Washer-Disinfectant Cleaning Monitor strip onto the Washer-Disinfectant Cleaning Monitor Record Sheet and complete the appropriate information. Retain as a permanent record.

STORAGE:

- Store Cleaning Monitors inside the protective foil bag provided at 40-86°F (4-30°C). Do not remove the monitors from the foil bag until ready to use.
- Keep away from disinfectants and fumes.

EXPIRY DATE:

The expiry date is printed on the product packaging.

Please see website for Washer-Disinfectant Cleaning Monitor Record Sheet and Troubleshooting Guide.

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